




**04510**

Please read the following instructions and warnings carefully.

Keep this instruction manual for future reference.

**For Technical Support Call 1-800-268-6237 or E-Mail at [customerservice@summerinfant.com](mailto:customerservice@summerinfant.com)**

For use with children from birth and up.

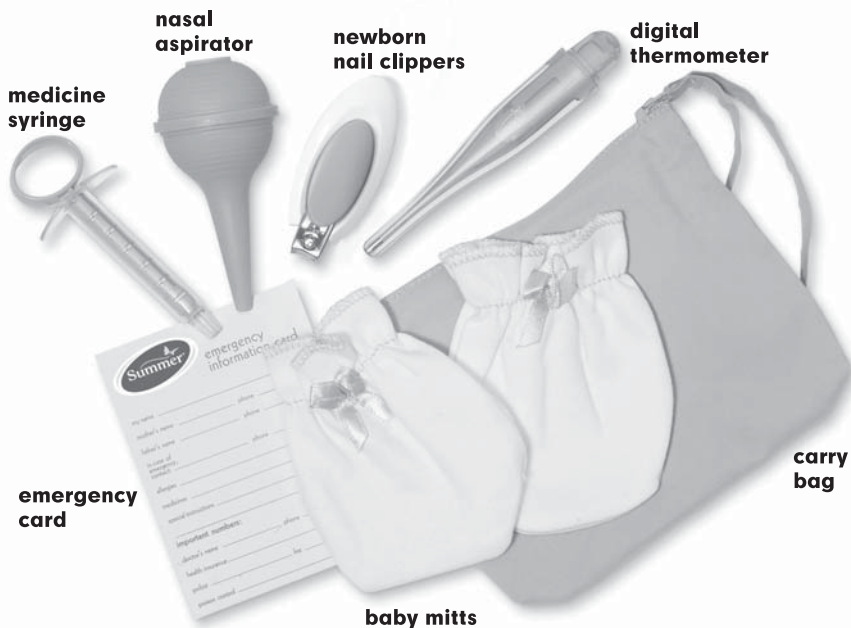

## INSTRUCTION MANUAL

### **DELUXE BABY'S ESSENTIALS KIT**

Thank you for buying the Baby's Essentials Kit from Summer Infant. This comprehensive kit contains all the essentials needed when you bring your baby home from the hospital. Convenient carry bag keeps these items organized and easily accessible.

### **⚠ CAUTION:**

The components of this product are not toys. Do not allow children to play with them.

---

## DIGITAL THERMOMETER:

---

For complete instructions for use and care, please see separate instruction manual enclosed.

---

## NASAL ASPIRATOR:

---

**To use:**

- Hold baby upright.
- While squeezing the aspirator bulb, gently place the tip of the aspirator into baby's nostril and tilt it to the side. Never point the tip directly in the center of the nostril. Be careful not to over-insert the aspirator.
- Slowly release the bulb allowing the suction to draw the mucus out of baby's nostril.
- Repeat with other nostril.
- Repeat as necessary until baby's experiences relief from congestion.
- To clean, put the aspirator under soapy, warm water. Squeeze and release the bulb several times to clean the inside of the bulb. Rinse with warm water. Squeeze out all excess water from the bulb.

---

## MEDICINE SYRINGE:

---

**To use:** Clean before first use. Push the inner half of the syringe all the way in and place the tip of the syringe into the liquid medicine. Pull the inner half out until the medicine reaches the correct marked dosage. Make sure your child is elevated to avoid choking and then place one finger inside their mouth and gently pull the cheek outward. Place the syringe toward the back of the cheek (not toward the back of the mouth to avoid gagging) and push down on the inner plunger to dispense the liquid.

**To clean:** Remove the inner plunger and wash both halves in warm, soapy water. Air dry. Top rack dishwasher safe. Do not place in boiling water.

---

## NEWBORN NAIL CLIPPERS:

---

This newborn nail clippers are for trimming baby's tiny fingernails and toenails. The rounded tips provide an additional measure of safety. Soft over-molded handle is comfortable and helps you maintain a steady grip.

**Note:** Do not clip too close to baby's skin. We recommend clipping baby's nails while he or she is sleeping.

**To clean:** Sterilize with isopropyl rubbing alcohol and rinse in warm water. Wipe dry.

---

## EMERGENCY CARD:

---

- Keep important phone numbers and medical information handy for quick reference.
- Keep in a safe place. Familiarize family members, baby-sitters and care givers with the location of the card and the information on it.

---

## BABY MITTS:

---

These little mitts will prevent your baby from accidentally scratching his face. Soft 100% cotton is machine washable.

**Note:** These mitts are designed to snugly fit infants up to 4 months old. Do not use on babies older than 4 months.

**To clean:** Wash before first use and frequently thereafter. Turn mitts inside out. Machine wash cool. Tumble dry low. No bleach.

---

## CARRY BAG:

---

- Keep items organized with the convenient storage case.

**To clean:** Wash with mild soap and warm water. Rinse thoroughly; air dry.

We at Summer Infant stand behind all of our products. If you are not completely satisfied or have any questions, please contact our Customer Service Department at 1-800-268-6237 or e-mail us at [customerservice@summerinfant.com](mailto:customerservice@summerinfant.com).

**Summer Infant, Inc.**

6 Blackstone Valley Place, Suite 206  
Lincoln, RI 02865 USA 1-800-263-6237

**[www.summerinfant.com](http://www.summerinfant.com)**

© 2005 Summer Infant, Inc.  
Please retain information for  
future reference.

**Summer Infant Europe, LTD.**

20 Sparrows Herne  
Bushey, Hertfordshire  
WD23 1FX UK 44 (0)20 8420 4429  
Colors and styles may vary.

**MADE IN CHINA**

6/05