



04591

Please read the following instructions and warnings carefully.

For Calpol, please refer to manufacturer's guidelines before use

Keep this instruction manual for future reference.

**For Technical Support Call (0)20 8420 4429 or E-Mail at [customerservice@summerinfant.com](mailto:customerservice@summerinfant.com)**

Calpol suitable from 3 months. Refer to manufacturers instructions.

## INSTRUCTION MANUAL

### COLD CARE KIT

Thank you for buying the Cold Care Kit from Summer Infant. This comprehensive kit contains essential health items for your baby. Convenient carry bag keeps these items organized and easily accessible.

#### NASAL ASPIRATOR:

**To use:**

- Hold baby upright.
- While squeezing the aspirator bulb, gently place the tip of the aspirator into baby's nostril and tilt it to the side. Never point the tip directly in the center of the nostril. Be careful not to over-insert the aspirator.
- Slowly release the bulb allowing the suction to draw the mucus out of baby's nostril.
- Repeat with other nostril.
- Repeat as necessary until baby's experiences relief from congestion.
- To clean, put the aspirator under soapy, warm water. Squeeze and release the bulb several times to clean the inside of the bulb. Rinse with warm water. Squeeze out all excess water from the bulb.

#### MEDICINE SYRINGE:

Easily dispense and measure medicine.

**To use:** Clean before first use. Push the inner half of the syringe all the way in and place the tip of the syringe into the liquid medicine. Pull the inner half out until the medicine reaches the correct marked dosage. Make sure your child is elevated to avoid choking and then place one finger inside their mouth and gently pull the cheek outward. Place the syringe toward the back of the cheek (not toward the back of the mouth to avoid gagging) and push down on the inner plunger to dispense the liquid.

**To clean:** Remove the inner plunger and wash both halves in warm, soapy water. Air dry. Top rack dishwasher safe. Do not place in boiling water.

### CAUTION:

The components of this product are not toys.  
Do not allow children to play with them.

nasal aspirator



medicine spoon



medicine syringe



forehead thermometer & case



#### MEDICINE SPOON:


**To use:** Clean before first use. Hold the Medicine Spoon upright and pour the correct dosage of liquid medicine into the handle, using the dosage markings for reference. To help ensure accuracy, keep the medicine spoon upright and at eye level. Make sure your child is sitting up, place the spoon in the child's mouth and gently tilt it upward.

**To clean:** Wash in warm, soapy water. Air dry. Top rack dishwasher safe. Do not place in boiling water.

#### FOREHEAD THERMOMETER STRIP & CASE:

This temperature indicator is reusable and comes with a convenient carrying case. The parent simply holds it in place on the child's forehead and gets a reading in less than 15 seconds.

We at Summer Infant stand behind all of our products. If you are not completely satisfied or have any questions, please contact our Customer Service Department at 44 (0)20 8420 4429 or e-mail us at [customerservice@summerinfant.com](mailto:customerservice@summerinfant.com).

 Forehead thermometer strip produced by:  
Hallcrest, Inc.  
0434 Glenview, IL USA

**Summer Infant, Inc.**  
6 Blackstone Valley Place, Suite 206  
Lincoln, RI 02865 USA 1-800-268-6237  
© 2005 Summer Infant, Inc.  
Please retain information for future reference.



visit us at  
[www.summerinfant.com](http://www.summerinfant.com)

**Summer Infant Europe, LTD.**  
20 Sparrows Herne  
Bushey, Hertfordshire  
WD23 1FX UK 44 (0)20 8420 4429  
Colors and styles may vary.  
**MADE IN CHINA** 9/05